


HOW TO USE MEDIPRO CHLOR-TABS

Ainsworth Medipro Chlortabs provide just over 1000ppm of available chlorine when diluted in 1 litre of cold clean water, and is effective for Terminal Cleans, Isolation Cleans, High Cleans and Infection Outbreaks.



Always wear suitable PPE such as gloves, apron and eye protection. If in a confined space or if prone to respiratory irritation then also wear a mask

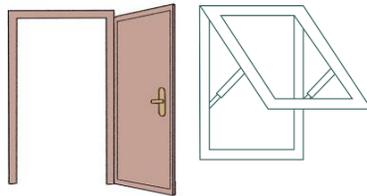

Improve room ventilation where possible

Instructions and SDS



Refer to safety information and check expiry date on tub.



Fill mixer bottle to shoulder/line with 1 litre of cold clean water – Add 1 tablet to water, screw on cap but leave the flip lid open and allow 6 minutes for the tablet to fully dissolve.

- **Allow 6 minutes for the tablet to dissolve**
- **Make sure the area is well ventilated and wear appropriate PPE**
- **Always use the diluter bottles – Don't prepare in a mop bucket**
- **In the event of large quantities of organic matter (especially vomit), remove with cloth or dustpan prior to disinfection**
- **Using Chlortab solution, scrub / wipe surfaces thoroughly that are to be cleaned and disinfected**
- **Leave a wet film for 6 minutes and allow to air dry**
- **Make a fresh solution at the start of each terminal clean**
- **Product expires after 24 hours**

NEVER

Use in a spray bottle

Apply to acids such as vomit

Mix with any other chemical or cleaning agent

Dilute or mix using hot or warm water



Ainsworth Medical
Since 1933

P 1300 602240

www.ainsworthmedical.com.au

PO Box 5055 Marrickville 1475